



# Rehabilitation Management System



**SCHILLER**  
The Art of Diagnostics

# Rehabilitation system

SCHILLER's SRMS (SCHILLER Rehabilitation Management System) was created with the belief that technology should allow you to increase contact with your patients and therefore accelerate the patient's recovery.

The quality guidelines for cardiac rehabilitation put high requirements on the systems used. Therefore, the acquisition and storage of full-disclosure data is of particular importance. From chip card-assisted creation of individual training protocols and reliable patient tracking to the automatic control of the connected ergometers and treadmills as well as the comprehensive monitoring of the training sessions, there is no need to accept compromise.

The tedious manual steps involved in creating required reports are eliminated. SRMS is a Windows-based application featuring familiar icons and an intuitive interface, allowing you to easily enter patient data, manage exercise sessions and complete patient outcome reports. The system is highly configurable to meet the needs of your rehab program, allowing you to add extra monitoring stations as class sizes increase, or an administration workstation dedicated to updating charts.


## The training, service and support you would expect from an industry leader

Your job relies on having accurate and dependable information. You need a system you can rely on. Your purchase of our system includes service training at the time of installation, so your team feels completely comfortable with every aspect of the new system. We also provide follow-up training within six months to ensure that you get the most out of your new system.



### Advantages of our SCHILLER Rehabilitation Management System

- High-resolution display that can be increased by double-clicking
- Up to 16 ergometers simultaneously (8 ergometers per monitor)
- Time-saving patient management
- Group assignment
- Single occupancy per chip card
- Modern medical training methods
- Pulse- and load-controlled training programs
- Treadmill programs
- IDE training program

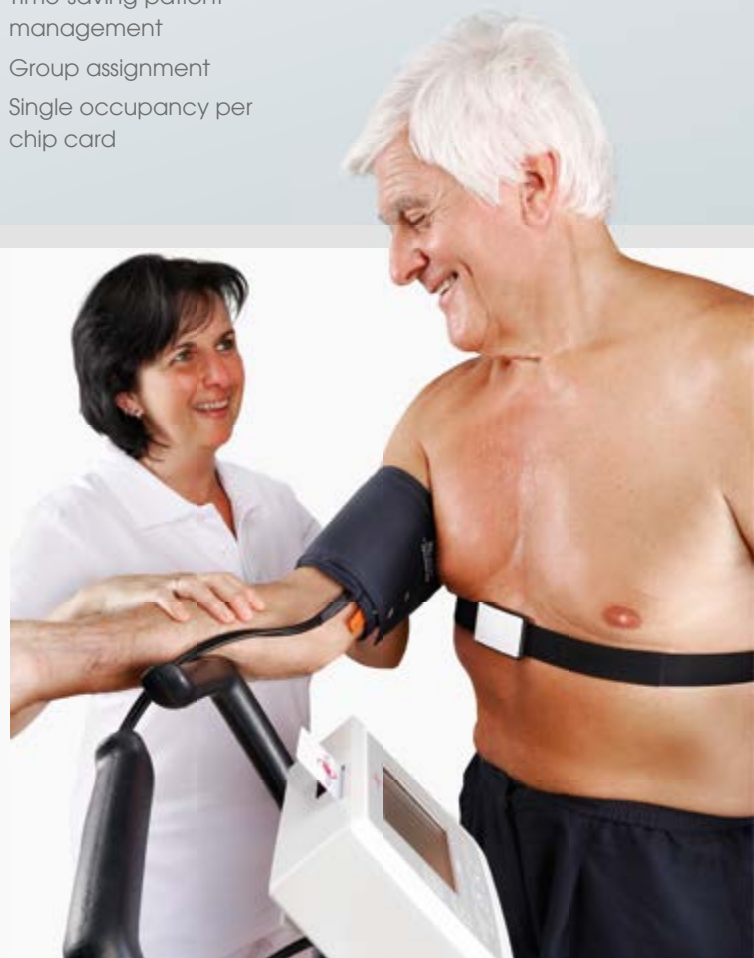



- Intuitive configuration of trainings
- Change of training parameters during training
- Diagnosis and documentation module
- Superimposition of training sessions (initial/final training)
- Alphanumerical evaluation table, all important measurements at a glance
- Rhythm protocol with zoom function
- All the data is compressed and saved in an SQL database

## ERG 911 ergometer

Clinicians around the world rely on our ergometer. Your patients will appreciate the solid and secure feeling of the ERG 911.

- The modular rehabilitation ergometers of the SCHILLER product range guarantee utmost safety and trouble-free operation, even under continuous load.
- The ergonomic design along with the dual adjustment capability for the handlebar (height and angle), guarantee the ideal seating position for any body height.
- The ergometer is optionally available with a saddle that can be electronically adjusted in height. This allows even elderly patients with lower extremity impairment to mount the ergometer without any problems.

- The wireless ECG amplifier (optional ECG chest belt) records an interference-free ECG.
- Many more options and accessories, from automatic blood pressure measurement to SpO<sub>2</sub> and adjustable pedal cranks are available to configure exercise systems for your requirements.

We at SCHILLER can offer you special ergometer and treadmill designs. They are compatible with our SRMS and include:

- the hand crank ergometer ERG 911 HKE that can be equipped with a wheelchair mount.
- the seat ergometer ERG 911 Comfort, suitable for adipose patients (up to 300kg).
- treadmill



**Asia**  
**SCHILLER Asia-Pacific / Malaysia**  
52200 Kuala Lumpur, Malaysia  
Phone +603 6272 3033  
Fax +603 6272 2030  
sales@schiller.com.my  
www.schiller-asia.com



**Austria**  
**SCHILLER Handelsgesellschaft m.b.H.**  
A-4040 Linz  
Phone +43 732 709 90  
Fax +43 732 757 000  
sales@schiller.at  
www.schiller.at



**China**  
**Alfred Schiller Medical Equipment Co. Ltd.**  
100015 Beijing, China  
Phone +86-10-84565453  
info@schillermedical.com  
www.schillermedical.cn



**France**  
**SCHILLER Médical S.A.S.**  
F-67162 Wissembourg/Cedex  
Phone +33 3 88 63 36 00  
Fax +33 3 88 94 12 82  
info@schiller.fr  
www.schiller-medical.com



**France (distribution France)**  
**SCHILLER France S.A.S.**  
F-77600 Bussy St Georges  
Phone +33 1 64 66 50 00  
Fax +33 1 64 66 50 10  
infoschiller@schiller-france.fr  
www.schiller-france.com



**Germany**  
**SCHILLER Medizintechnik GmbH**  
D-85622 Feldkirchen b. München  
Phone +49 89 62 99 81-0  
Fax +49 89 609 50 90  
info@schillermed.de  
www.schillermed.de



**Hungary**  
**SCHILLER Diamed Ltd.**  
H-1141 Budapest  
Phone +36 (1) 383-4780 / 460-9491  
Fax +36 (1) 383-4778  
sales@schiller.at  
www.schiller-hungary.hu



**India**  
**SCHILLER Healthcare India Pvt. Ltd.**  
Mumbai - 400 001, India  
Phone +91 22 6152 3333/ 2920 9141  
Fax +91 22 2920 9142  
sales@schillerindia.com  
www.schillerindia.com



**Croatia**  
**Schiller medicinski instrumenti d.o.o.**  
10000 Zagreb  
Phone +385 1 309 66 59  
Fax +385 1 309 66 60  
info@schillerzg.hr  
www.schiller.ch



**Latin America**  
**SCHILLER Latin America, Inc.**  
Doral, Florida 33172  
Phone +1 954 673 0358  
Fax +1 786 845 06 02  
sales@schillerla.ch  
www.schillerla.ch



**Poland**  
**SCHILLER Poland Sp. z o.o.**  
PL-02-729 Warszawa  
Phone +48 22 8432089  
Fax +48 22 8432089  
schiller@schiller.pl  
www.schiller.pl



**Russia & C.I.S.**  
**SCHILLER AG Rep. office**  
125124 Moscow, Russia  
Phone +7 (495) 970 11 33  
Fax +7 (495) 970 11 33  
mail@schiller-ag.com  
www.schiller-cis.com



**Serbia**  
**SCHILLER d.o.o.**  
11010 Beograd  
Phone +381 11 39 79 508  
Fax +381 11 39 79 518  
info@schiller.rs  
www.schiller.rs



**Slovenia**  
**SCHILLER d.o.o.**  
2310 Slovenska Bistrica  
Phone +386 2 843 00 56  
Fax +386 2 843 00 57  
info@schiller.si  
www.schiller.si



**Spain**  
**SCHILLER ESPAÑA, S.A.**  
E-28230-Las Rozas/Madrid  
Phone +34 91 713 01 76  
Fax +34 91 355 79 33  
schiller@schiller.es  
www.schiller.es



**Switzerland**  
**SCHILLER-Reomed AG**  
CH-8953 Dietikon  
Phone +41 44 744 30 00  
Fax +41 44 740 37 10  
sales@schiller-reomed.ch  
www.schiller-reomed.ch



**Turkey**  
**SCHILLER TÜRKIYE**  
Okmeydanı-Sisli - İstanbul  
Phone +90 212 210 8681 (pbx)  
Fax +90 212 210 8684  
sales@schiller-turkiye.com  
www.schiller-turkiye.com



**USA**  
**SCHILLER America Inc.**  
Doral, Florida 33172  
Phone +1 786 845 0620  
Fax +1 786 845 06 02  
sales@schilleramerica.com  
www.schilleramerica.com

03/14, EN



Article no.: 2.500654



**SCHILLER**  
The Art of Diagnostics