

DEFIGARD HD-7


Renewing defibrillation



DEFIGARD HD-7

Renewing defibrillation

Discover the DEFIGARD HD-7, SCHILLER's latest defibrillator / monitor. Compact and lightweight, benefit from its easy-to-use interface to access all main functions.

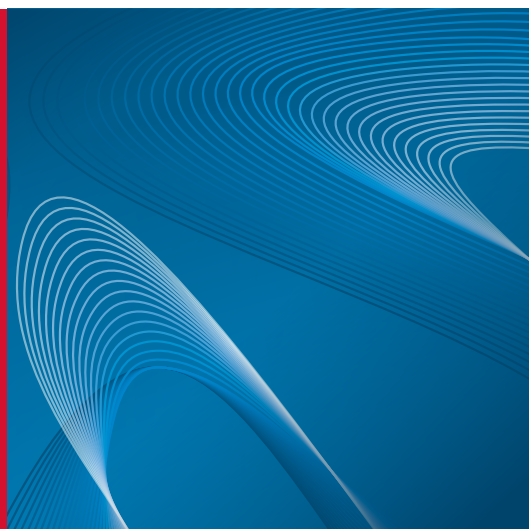

ALWAYS READY

The DEFIGARD HD-7's automatic self-tests are a time saver during routine device tests.

The DEFIGARD HD-7 provides paper printout but also supports paperless workflows.

Self-test results are automatically sent via a secure Wi-Fi connection to SEMA, SCHILLER's data management solution, for a quick and easy paperless review by healthcare professionals.

A LED clearly informs the user if the device is ready for use.


FAST TIME TO SHOCK

The DEFIGARD HD-7's fast charging time greatly minimises time to shock.


Unlike in a conventional AED, the DEFIGARD HD-7's cardiac rhythm analysis starts at the end of the CPR cycle which shortens the actual analysis duration as well as the "hands-off" time.

As the DEFIGARD HD-7 charges during the cardiac rhythm analysis, the shock is immediately available if it is recommended.

Conventional AED

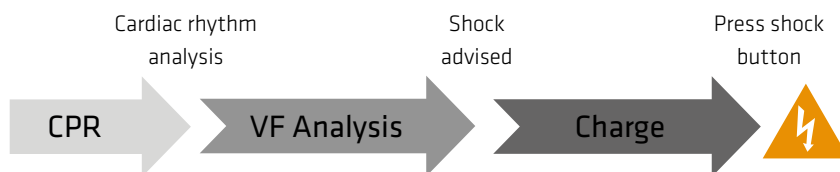

DEFIGARD HD-7's AED

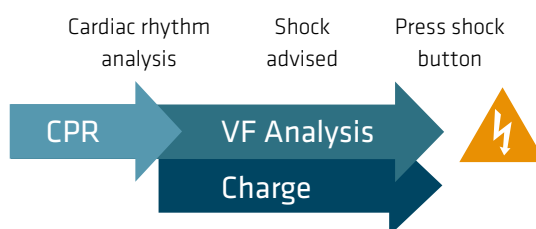



CPR FEEDBACK

The DEFIGARD HD-7 helps achieve highest efficiency in chest compressions:

- ❖ **Metronome feature:** delivers a steady rhythm to follow.
- ❖ **FreeCPR (optional):** provides feedback on the frequency of the chest compressions through the defibrillation pads, no additional sensor required.
- ❖ **ARGUS LifePoint 2 (optional):** provides feedback on the frequency, depth and recoil* of chest compressions. The DEFIGARD HD-7 indicates a color

coded visual assistance of the chest compressions during CPR. An audio assistance also guides the rescuer through the intervention.

The sensor ARGUS LifePoint 2 is shaped to perfectly fit the palm of your hand. It is made of durable rubber, is reusable and easy to clean.

*Recoil not available in the UK, France, Germany and USA.

COMPRESSION		
FREQ.	DEPTH	RECOIL
bpm	cm	
108	5.2	good
MESSAGE		
OK		

COMPRESSION		
FREQ.	DEPTH	RECOIL
bpm	cm	
153	5.4	good
MESSAGE		
PRESS SLOWER		

COMPRESSION		
FREQ.	DEPTH	RECOIL
bpm	cm	
153	4.2	good
MESSAGE		
PRESS DEEPER		



INTUITIVE DESIGN

The DEFIGARD HD-7's one-step mode selector allows intuitive and straightforward mode selection between AED, manual defibrillation, pacing and monitoring.

The user-friendly tactile *Touch'n' Save* interface allows direct access to the desired feature without losing time.

The DEFIGARD HD-7's interface has been developed to meet healthcare professionals' needs. For each selected mode, a global overview of patient vitals and needed actions is available.

Depending on lighting conditions, the user can adapt the display (high-contrast mode).



Manual defibrillation view



Pacer view



Monitoring view




AED view

A UNIQUE DATA MANAGEMENT SOLUTION

Transfer data to be reviewed and managed by SCHILLER's data management solution SEMA. Patient data is available continuously from outpatient care to intra-hospital care.

SEMA supports data management for the entire SCHILLER product range.

LifeDataNet G2 facilitates device pool management by deploying device configurations or software updates remotely.

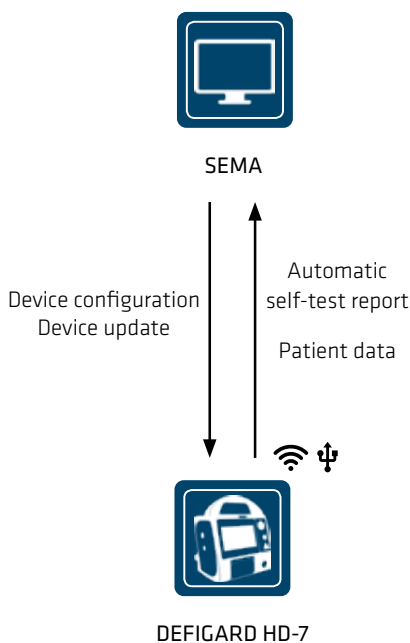

CYBERSECURITY

The DEFIGARD HD-7 ensures an extremely secure connection to the server using WPA2-Enterprise, the highest security standard in terms of wireless networks.

With the DEFIGARD HD-7, patients and their medical data are safe.

INNOVATIVE PADDLES

For additional flexibility, the DEFIGARD HD-7 is also available with defibrillation paddles, featuring:

- ❖ **Impedance LED:** check the patient's impedance at a glance via the illuminated LED
- ❖ **Key operating buttons:** energy selecting, charging and shocking can all be operated through the paddles' buttons
- ❖ **Paediatric mode:** simply remove the paddle's adult electrode plates to switch to paediatric mode. The device and energy levels will also adjust automatically.

INTERNAL DEFIBRILLATION

Internal defibrillation mode (optional) for open-heart surgical procedures, with a full range of spoon sizes and energy levels.



DEFIGARD HD-7

Matching all healthcare professionals' needs

INNOVATIVE PADDLES

Defibrillate with precision when using the paddles. Set the required energy level, check impedance, charge and defibrillate with the dedicated colour-coded buttons.

A UNIQUE EXPERIENCE

Equipped with a large 7" touchscreen, the DEFIGARD HD-7 benefits from a powerful Touch'n'Save interface and allows storage of intervention data.

CONNECTED DEVICE

Transfer intervention data swiftly to be reviewed and managed in SEMA, SCHILLER's data management solution. Easy fleet management through LifeDataNet G2.

INTEGRATED PRINTER

A large 3-channel built-in printer allows different printing configurations, amongst which the post-defibrillation report.

CAN BE MOUNTED ON A TROLLEY

The unit can be attached to the patient bed with hooks. A trolley is available (optional).





ALWAYS READY

The DEFIGARD HD-7 performs regular automatic self-tests and informs on its ready-to-use status through the blinking LED.

SIMPLIFIED USE

A one-step mode selector allows you to easily select the desired mode: semi-automatic AED, manual defibrillation, monitoring or pacing.

OPTIMISED USER INTERFACE

3 simple steps to follow for a clear overview during an intervention.

ACCURATE MONITORING

High-quality measurements:

- ❖ SpO₂ (optional): MASIMO or Nellcor™ technology
- ❖ 6-lead ECG: SCHILLER technology
- ❖ EtCO₂ (optional): MASIMO technology
- ❖ NIBP (optional): SCHILLER technology

SINGLE CONNECTOR

The DEFIGARD HD-7 has the same connector for pads, paddles and spoons.



Americas
SCHILLER Americas Inc.
 Doral, Florida 33172

North America:
 Phone +1 786 845 06 20
 Fax +1 786 845 06 02
 sales@schilleramericas.com
 www.schilleramericas.com

Latin America & Caribbean
 Phone +1 305 591 11 21
 Fax +1 786 845 06 02
 sales@schilleramericas.com
 www.schilleramericas.com



Asia
SCHILLER Asia-Pacific / Malaysia
 52200 Kuala Lumpur, Malaysia
 Phone +603 6272 3033
 sales@schiller.com.my
 www.schiller-asia.com



Austria
SCHILLER Handelsgesellschaft m.b.H.
 A-4040 Linz
 Phone +43 732 70 99 0
 Fax +43 732 757 000
 sales@schiller.at
 www.schiller.at



China
Alfred Schiller (Beijing) Medical Equipment Co., Ltd.
 100015 Beijing, China
 Phone +86 010 52007020
 Fax +86 010 52007020-8016
 info@schillerchina.com
 www.schillermedical.cn



Croatia
SCHILLER d.o.o.
 10000 Zagreb
 Phone +385 1 309 66 59
 Fax +385 1 309 66 60
 info@schillerzg.hr
 www.schiller.hr



France
SCHILLER Médical
 F-67160 Wissembourg
 Phone +33 3 88 63 36 00
 Fax +33 3 88 63 36 49
 info@schiller.fr
 www.schiller.fr



France (distribution France)
SCHILLER France S.A.S.
 F-77608 Bussy St Georges
 Phone +33 1 64 66 50 00
 Fax +33 1 64 66 50 10
 contact@schillerfrance.fr
 www.schiller-france.com



Germany
SCHILLER Medizintechnik GmbH
 D-85622 Feldkirchen b. München
 Phone +49 89 62 99 81 0
 Fax +49 89 609 50 90
 info@schillermed.de
 www.schillermed.de



India
SCHILLER Healthcare India Pvt. Ltd.
 Mumbai – 400 059, India
 Phone +91 22 6152 3333 / 2920 9141
 Fax +91 22 2920 9142
 sales@schillerindia.com
 www.schillerindia.com



Poland
SCHILLER Poland Sp. z o.o.
 PL-02-729 Warszawa
 Phone +48 22 843 20 89 / +48 22 647 35 90
 Fax +48 22 843 20 89
 schiller@schiller.pl
 www.schiller.pl



Russia & C.I.S.
AO SCHILLER.RU
 119049 Moscow, Russia
 Phone +7 (495) 970 11 33
 mail@schiller.ru
 www.schiller.ru



Serbia
SCHILLER d.o.o.
 11010 Beograd
 Phone +381 11 39 79 508
 Fax +381 11 39 79 518
 info@schiller.rs
 www.schiller.rs



Slovenia
SCHILLER d.o.o.
 2310 Slovenska Bistrica
 Phone +386 2 843 00 56
 Fax +386 2 843 00 57
 info@schiller.si
 www.schiller.si



Spain
SCHILLER ESPAÑA, S.A.
 ES-28232 Las Rozas/Madrid
 Phone +34 91 713 01 76
 Fax +34 91 355 79 33
 schiller@schiller.es
 www.schiller.es



Switzerland
SCHILLER-Reomed AG
 CH-8912 Obfelden
 Phone +41 44 744 30 00
 Fax +41 44 740 37 10
 sales@schiller-reomed.ch
 www.schiller-reomed.ch



Turkey
SCHILLER TÜRKIYE
 Okmeydani-Sisli – Istanbul
 Phone +90 212 210 8681 (pbx)
 Fax +90 212 210 8684
 info@schiller.com.tr
 www.schiller-turkiye.com



United Kingdom
SCHILLER UK Ltd.
 Anstruther, Fife KY10 3H
 Phone +44 16 190 72 565
 sales@schilleruk.com
 www.schilleruk.com

SCHILLER reserves the right to make changes in design, specifications and features shown or discontinue the product described at any time without notice or obligation. Images may be representative. Some products may not be regulatory cleared or released for sale in all markets. Contact your SCHILLER representative for the most current information and availability. Classification according to regulation (EU) 2017/745: III; Notified body: GMED CED459; Manufacturer: SCHILLER Medical; Please read carefully the user guide before using the DEFIGARD HD-7. This document is compliant with article R5213-2 of "Code de la santé publique".

